



PHILIPS
RESPIRONICS

Masks and
accessories

Technology that cares

Homecare masks and accessories

Mask types

Full face

Nasal

Pillow

Pediatric

Accessories

Sizing and Fitting

Practical info

Choosing the right type of mask



Nasal

Generally offered as the first option, nasal masks are generally smaller and lighter than full face masks. If higher pressure is needed, nasal masks could provide better comfort than pillow masks.



Pillow

A good solution if your patient feels claustrophobic in masks that cover a lot of the face, or if your patient has a deviated septum and/or facial hair.



Full face

Ideal if your patient suffers from nasal obstruction or frequent congestion due to allergies or cold symptoms. Also suitable for patients who continue to breathe through their mouth at night despite trying the combination of a nasal/nasal pillow mask with a heated humidity feature or chin strap.



Pediatric masks

Specifically designed for children, from infant (1 year old) to adolescents.



Amara View

Smallest and lightest*
of all leading full-face masks**

*without quick release tube

**Internal testing: data on file

Mask with headgear	With exhalation port	
	International ⁽²⁾	Global ⁽¹⁾
Small	1090662	1090602
Medium	1090663	1090603
Large	1090664	1090604

Innovative **under-the-nose** design
Prevents red marks on the bridge of the nose**



Headgear

Soft & comfortable with straps and crown strap

Large	1090696
Standard	1090697

Under-the-nose cushion

One modular frame for all cushion sizes

Small	1090692
Medium	1090693
Large	1090694

Quick-fit headgear clips

with adjustable tabs

Clips (x2)	1090698
------------	---------

Quick-release tube

Tube	1090689
------	---------



DreamWear

Like wearing nothing at all*

Fit Pack includes all 4 cushion sizes (S, M, L, MW) and 1 medium sized frame.

Mask

with headgear	International ⁽²⁾	Global ⁽¹⁾
Medium	1116720	1116700

*2015 Philips User Preference Questionnaire, data on file.

**Mask does not directly contact the bridge of the nose or nostrils.

top of head connection

gives your patient the freedom to move

Soft flexible frame

reduces facial irritation

Small	1116745
Medium	1116746
Large	1116747

Fabric wraps

provides a soft interface to the patient's cheek

Wraps (x2) 1116754

Slim-line headgear

Headgear 1116750

Under the nose nasal cushion

Prevents red marks, discomfort or irritation in the nostrils or on the nose bridge**

Under-the-nose cushion

Small	1116740
Medium	1116741
Large	1116742
Medium wide	1116743





Wisp

Performance has a new style

Each package contains three cushion sizes, which fit on the same frame.

Adult mask includes S/M, L and XL

Mask with headgear	With exhalation port	Without exhalation port
Adult fabric frame	1094061	
Adult silicone frame	1094060	1094062

Headgear with integrated tube holder

Small	1109307
Standard	1094082
Large	1094078

Reversible fabric frame
soft suede or silky sateen

Fabric	1094081
Silicone	1094080

Click fit headgear clips

Clips (x2)	1094083
------------	---------



360° rotating elbow

Tube & elbow	1105624
--------------	---------

Cushion

Small/med	1094087
Large	1094088
X-large	1112031

'Tip-of-the-nose' cushion design

featuring Auto Seal groove technology



Pico

Value-added performance

Mask with headgear	With exhalation port	
	International ⁽²⁾	Global ⁽¹⁾
Small/medium	1104921	1104915
Large	1104922	1104916
X-large	1104923	1104917

Slim-line **headgear** with stabilizing crown strap and integrated forehead support

Standard	1104934
Small	1104935



Cushion

Small/med	1104936
Large	1104937
X-large	1104938

Over the nose cushion

snap fit with Auto Seal technology

Quick-fit **headgear clips** with adjustable tabs

Clips (x2)	1104933
------------	---------

Quick disconnect **swivel**

Swivel	1104931
--------	---------



Wisp Youth

Performance has a new style

Each package contains three cushion sizes, which fit on the same frame. Youth mask includes P, S/M and L

Mask with headgear	With exhalation port
Youth fabric frame*	1109308

*Youth mask intended for both adult and pediatric use (7 years old or >18 kgs)

Headgear with integrated tube holder

Small	1109307
Standard	1094082

Reversible fabric frame soft suede or silky sateen

Fabric	1094081
--------	---------

Click fit headgear clips

Clips (x2)	1094083
------------	---------



Cushion


Petite	1094086
Small/med	1094087
Large	1094088

'Tip-of-the-nose' cushion design

featuring Auto Seal groove technology

360° rotating elbow


Tube & elbow	1105624
--------------	---------

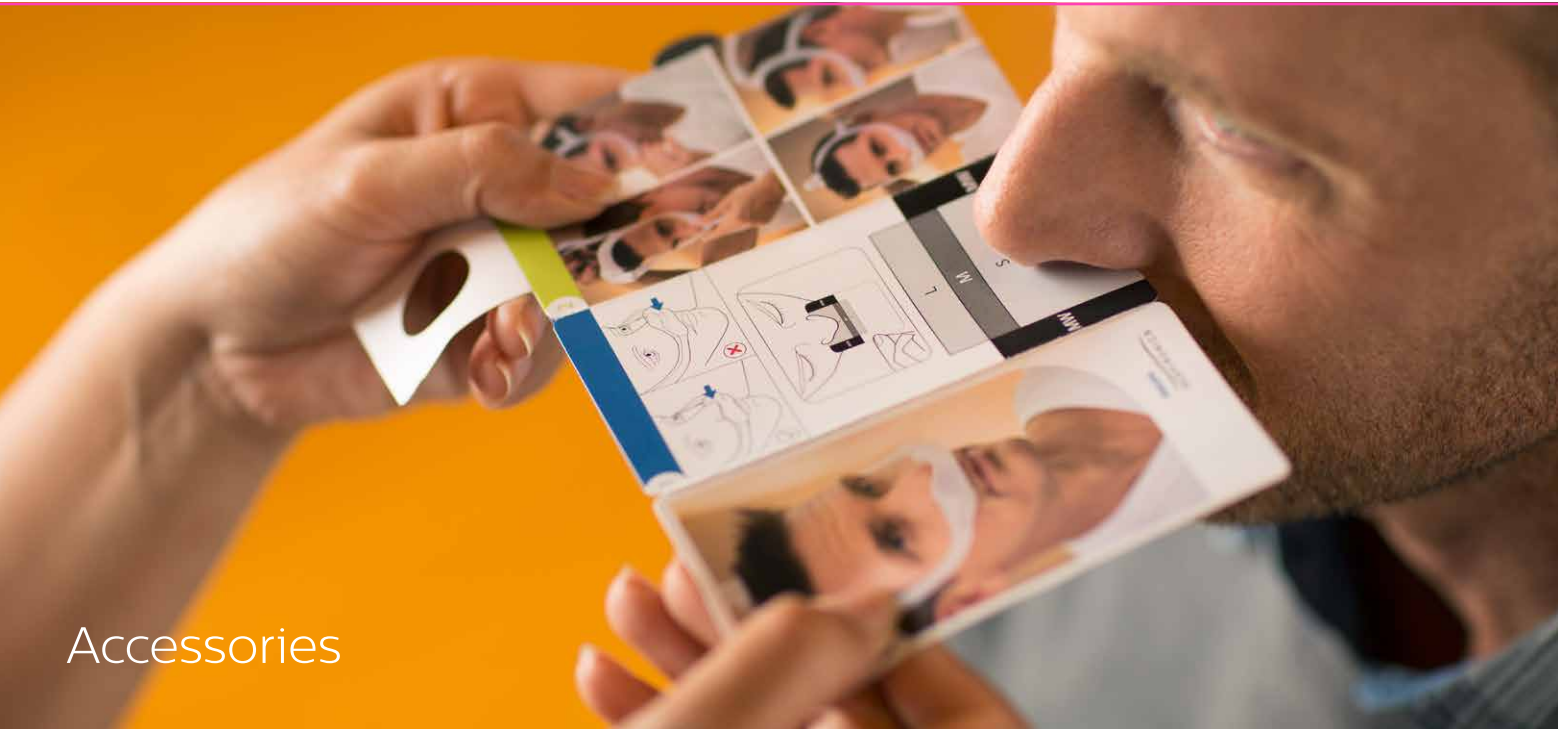

Accessories

Successful mask sizing


Sizing for **over-the-nose nasal** masks

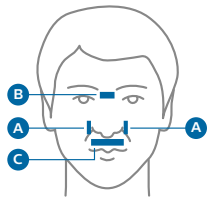

Facial landmarks


- a) Sides of nostrils
- b) Bridge of nose zone
- c) Below the nose tip, above the lip

For individual mask sizing and fitting steps, refer to the mask's user instruction guide.

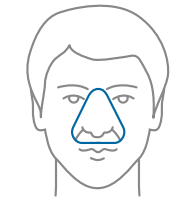

For ComfortGel Blue and EasyLife only.

Place the sizing gauge **over the nose**.

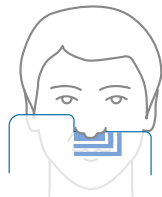


For True Blue, Pico and Wisp only.

Place the sizing gauge **under the nose**.

Step 1 (sizing) • Step 2 (fitting) • Step 3 (adjusting)


Sizing for **over-the-nose full face** masks

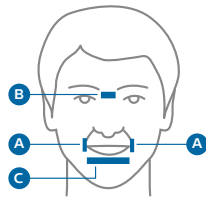

Facial landmarks


- a) Sides of mouth
- b) Bridge of nose zone
- c) Below the lower lip

When sizing an over-the-nose full face mask, ensure the mouth is slightly open.

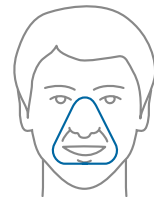

For ComfortGel Blue Full only.


Place the sizing gauge **over the nose**.

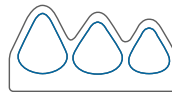

For Amara Gel only


Place the sizing gauge **next to the nose**.

Sizing guide for **under-the-nose nasal** and **full face** masks

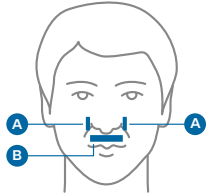

Facial landmarks

- a) Sides of nostrils
- b) Depth of nose

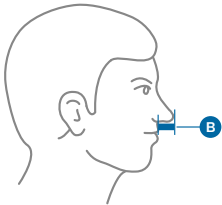

For DreamWear and Amara View only.

Place the sizing gauge **under the nose** **without pushing against the cheeks** and the tip of those nose will indicate whether your patient needs a small, medium or large cushion size.

Note: If your patient measures close to the threshold, always choose the smaller size.



Which device for which sleep therapy patient

💧 = Humidifier available in option 🌀 = Heated hose available



DreamStation CPAP Pro*

Patient type:

OSA

Modes:

- Fixed CPAP
- CPAP-check

Features:

- EZ-Start



DreamStation Auto CPAP*

Patient type:

OSA

Modes:

- Fixed CPAP
- Auto CPAP
- CPAP-check

Features:

- EZ-Start
- Opti-Start
- Auto-Trial



DreamStation Auto BiPAP*

Patient type:

OSA (severe or non compliant patient)

Modes:

- Fixed bi-level
- Auto bi-level
- Fixed CPAP

Features:

- EZ-Start



System One BiPAP autoSV Advanced

Patient type:

Complex breathing patterns such as CSR, CSA, mixed apnea, and complex apnea

Modes:

- Auto CPAP
- BiPAP S and S/T
- Auto CPAP + servo ventilation
- BiPAP + servo ventilation

* Regional availability may vary

Which device for which ventilation patient

💧 = Humidifier available in option 🌀 = Heated hose available



BiPAP S/T and BiPAP AVAPS

Patient type:

Chronic respiratory insufficiency patients (adult /children >18kg)

Modes:

- BiPAP S/T: CPAP, S, S/T, PC, T
- BiPAP AVAPS: CPAP, S, S/T, PC, T



BiPAP A30 and BiPAP A40

Patient type:

Chronic respiratory insufficiency patients (adult /children >10kg)

Modes:

- BiPAP A30: CPAP, S, S/T, PC, T
- BiPAP A40: NIV and IV capabilities, CPAP, S, S/T, PC, T, AVAPS-AE



Trilogy 100 and Trilogy 200

Patient type:

Chronic respiratory insufficiency patients (adult /children >5kg) life support

Modes:

- Invasive and non-invasive
- Volume modes: AC, CV and SIMV
- Pressure modes: CPAP, S, S/T, PC, T, PC-SIMV, AVAPS-AE, and MPV



(1) Global: Brazilian Portuguese, Dutch, English, French, German, Italian, Spanish.

(2) International: Arabic, Bulgarian, Croatian, Czech, Danish, Dutch, English, Estonian, Finnish, French, German, Greek, Hebrew, Hungarian, Italian, Kazakh, Latvian, Lithuanian, Macedonian, Norwegian, Polish, Portuguese, Romanian, Russian, Serbian, Slovak, Slovenian, Spanish, Swedish, Turkish, Ukrainian.

Not all products are available in all countries. Please check with your local Philips Respironics representative for availability

© 2015 Koninklijke Philips N.V. All rights reserved. Specifications are subject to change without notice.

Customer Service +1 724 387 4000

www.philips.com/respironics

1067563 4107102

PN 1119259 MCI 4107101